

RIDX™ Giardia Ag Test Kit

Giardia is a microscopic parasite that causes diarrheal disease. The variability in the concentration of organisms in the stool can make *Giardia* difficult to diagnose.

The RIDX™ Giardia Ag Test Kit is a diagnostic reagent that can detect *Giardia* cyst antigen quickly and simply within 10 minutes after dropping the sample.

Principal: Chromatographic immunoassay for the qualitative test
Specimen: Canine or feline feces

Components

Giardia Ag test device (10 tests)
Sample dilution buffer (10 vials)
Disposable swab (10 ea)
Disposable dropper (10 ea)

Features

Clinical Sensitivity 97.1% (34/35)






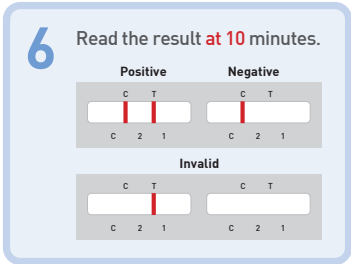
Clinical Specificity 96.5% (109/113)

Limit of Detection 125 *Giardia* cysts/100ul

No Cross-reactivity with other pathogens (CCV, CDV, CIV, FCoV, and FPV)

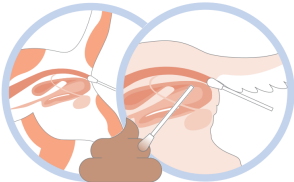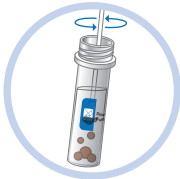

Test Procedure

- Using a swab to collect diarrhea samples or diarrhea feces in the anus.
- Put the swab into the sample dilution buffer and stir the sample with the swab.
- Remove the swab from the sample dilution buffer.
- Place the tube until the large particles down.
- Take the supernatant using a dropper. Add 4 drops of the sample solution into the sample hole on the device.
- Read the result at 10 minutes.

CAT No.	Description	LIC No.	Type	Package size
CGM-VGG-11	RIDX Giardia Ag Test Kit	300-001	LFA*	10 Tests/Kit

* LFA: Lateral flow assay